

How to Use *CheckUp*



1

Pour the litter

Pour all hydrophobic litter in a clean and empty litter box.



2

Wait for your cat

Allow your cat to urinate freely on the hydrophobic litter.



3

Collect samples

Collect urine from the top of the sand with the pipette.



4

Fill the vial


Fill the vial at least 3/4 way using the pipette.



5

Dip the strip

Simply dip the strip into the vial and make sure all squares are submerged.

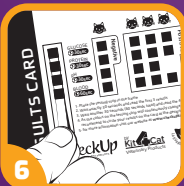

6

Read results

Quickly place the strip next to the Results card and follow the instructions underneath the table.



7

Keep the sample

If necessary, close the vial with the cap and keep in the refrigerator.



8

Further analysis

If test shows any positive result, take the sample to your vet for further testing.

⚠ Important Notes!

- Before pouring the hydrophobic litter, make sure the litter box is clean and empty.
- It is extremely important to read the results according to the outlined time frames.
- Please note that the colors on the diagnostic strip will continuously change.
- Sample can be refrigerated for up to 48 hours before being taken for further evaluation.
- CheckUp is a single-use product and should not be reused.

RESULTS CARD

Place Your Tester Here

GLUCOSE
⌚ 30sec

PROTEIN
⌚ 30sec

pH
⌚ 30sec

BLOOD
⌚ 60sec

Negative

	NEGATIVE(-)
	NEGATIVE(-)
	5.0 6.0
	NEGATIVE(-)

	100 5		250(±) 15		500(++) 30		1000(+++) 60		>2000(++++) 110	mg/dL mmol/L
	TRACE(±)		30(+) 0.3		100(++) 1.0		300(+++) 3.0		>2000(++++) >20	mg/dL g/L
	6.5		7.0		7.5		8.0		8.5	
	NON-HEMOLYZED(±) UTRACE		HEMOLYZED(±) UTRACE		SMALL(+) 25		MODERATE(++) 80		LARGE(+++) 200 or	CELLS/μL

CheckUp
AT-HOME WELLNESS TEST FOR PETS

Coastline Global

1. Once you have dipped the testing strip into the vial of collected urine, immediately place it into the frame above.
2. After 30 seconds, read the first 3 results and circle the matching colors.
3. After waiting another 30 seconds (60 seconds since the dipping), read the 4th result and circle the matching color.

CAUTION: As the colors on the testing strip will continuously change over time, it is important to evaluate the colors at exactly 30 and 60 seconds from the dipping.

For optimal results, collect urine in the morning or 2 hours after a daytime meal.