

# How to Use *CheckUp*



## Pole assembly

Attach the metal ring to the telescopic pole by rotating it.



## Place the cup

Pull the metal ring to extend the pole and place the cup in it.



## Collect sample

Catch a sample while dog is urinating and bring home for testing.



## Fill the vial

Fill the vial at least 3/4 way using the pipette.



## Dip the strip

Simply dip the strip into the vial and make sure all squares are submerged.



## Read results

Quickly place the strip next to the Results card and follow the instructions underneath the table.



## Keep the sample

If necessary, close the vial with the cap and keep in the refrigerator.



## Further analysis


If test shows any positive result, take the sample to your vet for further testing.


## ⚠ Important Notes!

- To avoid contamination, make the sure cup is clean and seal it after collection.
- It is extremely important to read the results according to the outlined time frames.
- Please note that the colors on the diagnostic strip will continuously change.
- Sample can be refrigerated for up to 48 hours before being taken for further evaluation.
- CheckUp is a single-use product and should not be reused.

# RESULTS CARD

Place Your Tester Here









**GLUCOSE**  
⌚ 30sec

**PROTEIN**  
⌚ 30sec





















**pH**  
⌚ 30sec


**BLOOD**  
⌚ 60sec

**Negative**


	NEGATIVE(-)
	NEGATIVE(-)
	5.0 6.0
	NEGATIVE(-)

**Positive**


	100 5		250(±) 15		500(++) 30		1000(+++) 60		>2000(++++) 110	mg/dL mmol/L
	TRACE(±)		30(±) 0.3		100(++) 1.0		300(+++) 3.0		>2000(++++) >20	mg/dL g/L
	6.5		7.0		7.5		8.0		8.5	
	NON-HEMOLYZED(±) TRACE		HEMOLYZED(±) TRACE		SMALL(++) 25		MODERATE(+++) 80		LARGE(++++) 200 or more	CELLS/μL



AT-HOME WELLNESS TEST FOR PETS



Lab Validated & Vet Approved



1. Once you have dipped the testing strip into the vial of collected urine, immediately place it into the frame above.
2. After 30 seconds, read the first 3 results and circle the matching colors.
3. After waiting another 30 seconds (60 seconds since the dipping), read the 4th result and circle the matching color.

**CAUTION:** As the colors on the testing strip will continuously change over time, it is important to evaluate the colors at exactly 30 and 60 seconds from the dipping.

For optimal results, collect urine in the morning or 2 hours after a daytime meal.